



Saltpipe Refillable



Salt inhaler to help you breathe more easily and cleanse the respiratory system

Description

Portable ceramic inhaler containing salt crystals from the Transylvanian Praid Salt mine. The salt crystals are placed in a salt bag inside the ceramic inhaler. For hygienic reasons, the salt bag is replaceable and the entire device is cleanable. The active ingredients may last for up to 1 year under normal usage. Replacement salt bags are also available to order.

Recommended for

People suffering from asthma, respiratory allergies and other respiratory system disorders.

People who need assistance for recovery from respiratory illness, who need an aid to clean their respiratory system and would like to breathe more easily.

Claims

As supplementary treatment the Saltpipe Refillable can reduce the symptoms of

- ✓ Asthma
- ✓ Respiratory allergies (allergic asthma, allergic rhinitis)
- ✓ Chronic obstructive pulmonary disease (COPD)
- ✓ Residual respiratory problems after quitting smoking

Advantages

A handy, portable product that can be used anytime and anywhere. Curative properties of the Praid salt are well known and appreciated thanks to the existing therapeutic treatment facilities in the abandoned parts of the mine.

Instructions for use

Simply place your mouth over the mouthpiece and breathe in normally through the mouth and exhale through the nose. Prolonged inhalation is not necessary. For best results use the inhaler every day. Recommended use: 15 to 25 minutes per day. Suitable for children from 5 years old and adults.

How it works

When inhaling, the moisture from the air passing the salt crystals absorbs micron sized salt particles, which then enter the respiratory system with each intake of breath, bringing the beneficial effects of natural salt therapy. The inhaled salt stimulates the natural cleaning mechanisms, which flush away any impurities from the surface of cells lining the lungs

Packaging contains

Saltpipe Refillable: salt inhaler (salt bag included), user manual.

Refill Bag: salt bag, user manual

Documentation and others

EC Declaration of Conformity, EC Certificate (in compliance with the Council Directive 93/42/EEC), ISO 13485 Certificate, ISO 9001 Certificate

Clinical evaluation and tests

5 min commercial spot

Wholesale Inquiries

All level of wholesale inquiries are welcome. To discuss further details, please [contact us](#).

Payment Terms: on delivery

Country of Origin: European Community

Pick-up Point: Budapest

Delivery Term, Currency and Price: [ask for quote](#)

Minimum Order Quantity: [ask for quote](#)

Package Details

Saltpipe Refillable

1) inner box with graphic design, 2) outer box to protect the ceramic inhaler

Product Dimension (cm): 12 x 6.5 x 6 cm

Master Carton Quantity: 100 pcs

Product Net Weight (kg): 0.2 kg

Master Carton Dimensions (cm): 34 x 33 x 50 cm

Product Gross Weight (kg): 0.23 kg

Master Carton Gross Weight (kg): 24 kg

Refill Bag

Product Dimension (cm): 6.2 x 5.6 x 5.5 cm

Master Carton Quantity: 60 pcs

Product Net Weight (kg): 0.08 kg

Master Carton Dimensions (cm): 40 x 31 x 12 cm

Product Gross Weight (kg): 0.09 kg

Master Carton Gross Weight (kg): 8 kg

product category: class IIa medical device

website: www.oxfordmedicals.com/saltpipes.html

 2409